

Easyboot Glove Application



The Easyboot Glove is a lightweight, slim profile, form-fitting boot that is easy to apply. It was designed to fit like a second skin over the hoof without adding bulk or width. This seamless boot hugs the hoof and responds like a natural foot. Flexible and tough, the Easyboot Glove adds traction and protection to the hoof, while allowing a free stride and natural breakover. Like a glove, this boot provides protection without hindering mobility.

The Easyboot Glove material stretches over the hoof and clings to the wall so that debris stays out of the boot even through sandy and muddy conditions. No external hardware means that there is no need to worry about grass or sticks getting caught in the boots. No hardware problems, no cables or buckles to repair. Studs are available for winter riding.

Easyboot Glove Application

- 1. Pick up the horse's leg** and support the cannon bone between your knees so that you can use two hands when applying the boot.
- 2. Fold the gaiter back to get it out of the way.**



3. Using technique rather than strength, place the boot over the horse's toe and, using a slight back and forth twisting motion, slide the boot on the rest of the way. You can also put the foot down on the ground and allow the horse to put weight into it. Don't use the gaiter to pull the boot on. If you are having trouble please refer to the fitting video found at easycareinc.com.


4. Check for proper fit. Make sure all portions of the boot (with the exception of the gaiter) are below the hairline. The boot should be very snug and not twist on the hoof. Do not use the boot if it is loose or if the main part of the boot touches the coronary band.

- 5. Fasten the Easyboot Glove Gaiter.**
- Pull the top of the gaiter tight around the pastern.
 - Fasten the hook and loop straps.

The tighter the fit, the better the boot will perform. Overlap the hook and loop straps as much as needed to insure a tight fit and avoid rubbing issues. (See illustration below)

This overlapping does not compromise the integrity of the gaiter. When properly tightened, it should be difficult to fit a finger between the pastern and the top of the gaiter.

Due to their great deal of elasticity, gaiter straps cannot be over tightened.

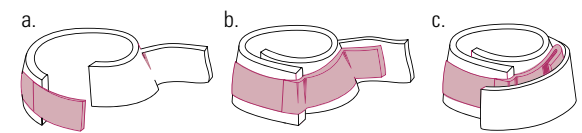

Removing the Easyboot Glove.

- Unfasten the gaiter straps.
- Slip the boot off the hoof.



Have Questions? Need Help? Visit easycareinc.com



EasyCare
easycareinc.com
1.800.447.8836

EasyCare Inc. Warranty

EasyCare offers a 90 day repair or replacement warranty for all hoof boots (purchased as new) manufactured under the EasyCare label. Please register your product at http://www.easycareinc.com/warranty_reg.aspx and keep your original dated receipt(s) as proof of purchase during the warranty period.

If a boot fails during warranty, it is generally a minor hardware issue. Please bear in mind that most components of EasyCare hoof boots are either replaceable or repairable. This includes wire laces, cables, buckles, gears, rollers, screws and straps. Most repairs can be easily accomplished with simple household tools. Depending on the circumstances, EasyCare will send you repair parts, or ask that you return the boots to us for evaluation. Videos of many repairs are available on our website at www.easycareinc.com.

While this warranty is very broad in nature, it does not cover wear (including sole breakthrough when worn over horseshoes), misuse, abuse, factory seconds, bargain bin items, or boots purchased as used. Products returned for evaluation must be clean, dry and free of debris, dirt, sand and manure. *Failure to return clean products will result in product being shipped back at sender's expense or a cleaning fee may be assessed.* Damage inflicted by omnivorous horses, rodents or other unusual abuse is not covered.

Returns must be authorized before sending and must be returned with an RMA number. Contact EasyCare directly for all warranty issues at 1-800-447-8836 to receive an RMA number.

Specifically:

1. Returns for credit only (refund of purchase price only): Products must be in new, unused condition, free of dirt and wear marks. It is your responsibility to evaluate the fit of your new product in a manner where you may return it in new condition if you have doubts about the fit. Be certain that you know the return policy of product purchased from dealers other than direct from EasyCare (the manufacturer). EasyCare will not give credit for product purchased elsewhere. Return to point of purchase in that circumstance. Also, all brochures, inserts and/or free gifts must accompany returned items.


Online Warranty Registration:
www.easycareinc.com/warranty_reg.aspx

- Exchanges of new product (size or color issue): EasyCare will only exchange NEW items with same product style and model for same product when requested with NEW items only. You may exchange for a different size or color exchange only, no exceptions. If you wish to have a different style of hoof boot, etc., you must return new item for refund (original point of purchase) and then purchase new choice.
- Warranty failure (used product that is within 90 days of purchase): Contact us at 800-447-8836. EasyCare will either provide repair parts directly to you, or repair/replace after our evaluation. Items must be returned to EasyCare at the consumers cost with original receipt. EasyCare will return the items via ground service (UPS or FEDX) free of charge. Requests for faster shipping service will be at the expense of the consumer.
- Not under warranty (over 90 days or no proof of purchase): Repair parts may be purchased from your local dealer or EasyCare, or product may be returned for evaluation and possible repair (parts, labor and freight charges will apply).
- Call for an RMA number (under warranty or not): Returns must be authorized before sending and must be returned with an RMA number clearly written on outside of packaging. Contact us at 800-447-8836 to receive an RMA number. Returns will be refused without an RMA number on outside of package.
- Freight charges (outside warranty period): Freight charges to and from EasyCare for repairs, exchanges and items outside the warranty period will be the purchaser's responsibility.
- EasyCare staff is always willing to discuss any specific issues you may have regarding the performance of your EasyCare or Old Mac's products. Please contact us at 800-447-8836 or email to admin@easycareinc.com. Visit our website, www.easycareinc.com, for tips and product information that will help with the use of any of our product line.



How to Measure for Boots

- After a fresh trim, measure the width of the hoof across the bottom at the widest point.
- Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

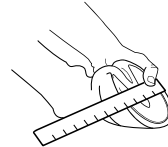

Measuring for
Hoof Width



Measuring for
Hoof Length

- Compare your measurements with the appropriate size chart. The length and width measurements must fit into the same size on the chart. Because this boot needs to fit very snug, if you are a size apart we do not recommend this boot. Be sure to check the requirements and measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.

| Size | Width in mm | Length in mm |
|------|-------------|--------------|
| 00 | 96-99mm | 106-109mm |
| 00.5 | 100-103mm | 110-113mm |
| 0 | 104-107mm | 114-117mm |
| 0.5 | 108-111mm | 118-121mm |
| 1 | 112-115mm | 122-125mm |
| 1.5 | 116-119mm | 126-129mm |
| 2 | 120-123mm | 130-133mm |
| 2.5 | 124-127mm | 134-137mm |
| 3 | 128-131mm | 138-141mm |
| 3.5 | 132-135mm | 142-145mm |
| 4 | 136-139mm | 146-149mm |

Success of the Easyboot Glove and Easyboot Glue-On are determined by accurate measurements and a snug fit.

Do not assume that, if you wear a certain size in another boot style, you will wear the same size Glue-On or Glove.

We have found that measurements are more accurate and easier to take using the metric system.

We continue to strive for the best products on the market with you and your horse in mind. We hope you enjoy our product selection and website improvements. Thank you for your continued support and patronage.

The Fine Print:

EasyCare Inc. hereby disclaims responsibility for the misuse of any products described in this brochure and web site and/or losses incurred through the use of products by any method other than that which is outlined in this catalog or web site or EasyCare's product packaging. All products should be used in accordance with the instructions printed on the product's label. EasyCare Inc products are not intended to treat, cure or diagnose any medical condition.

www.easycareinc.com 800.447.8836



Easyboot® Glove

Application Guide